




what is reyoungel

Reyoungel Esthetics Products with pure hyaluronic acid gelstabilized for long duration. Reyoungel is a injectable gel made of no-animal cross-linked hyaluronic acid . They offer a complete solution for treating facial lines, wrinkles, and lips . It can also be used in facial sculpting by enhancing natural facial contours and features . There are three types of products can be Chosed, Reyoungel Derm , Reyoungel deep and Reyoungel Fine . This three products make it possible to quickly and effectively meet the vast majority of patient's needs for correction and augmentation of wrinkle and folds.



- 1 Reyoungel Derm-for treatment of wrinkles and lip enhancement.
- 2 Reyoungel Fine-for correction of thin lines, across the forehead and around the eyes and mouth.
- 3 Reyoungel Deep-for correcting deep folds, creating fuller lips and shaping facial contours, such as cheeks and chin.

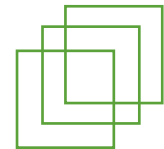

Reyoungel Esthetics Products

The first step to successful treatment is selecting the right Reyoungel Esthetics product for a specific indication. Each product is based on our clear Non-Animal Stabilized Hyaluronic Acid gel, the distinguishing factor is gel-particle size.

Reyoungel Fine

With the smallest particles for the correction of very thin, superficial lines. Injected in the upper part of the dermis.

Reyoungel Derm

Medium particle size for moderate wrinkles. Injected in the middle part of the dermis.

Reyoungel Deep

Larger particles for deep folds . Injected in the deep layer of the dermis and or the surface layer of the subcutis.



Products illustration

	REYOUNGEL DEEP	REYOUNGEL DERM	REYOUNGEL FINE
Composition	24mg/ml stabilized hyaluronic acid	24mg/ml stabilized hyaluronic acid	24mg/ml stabilized hyaluronic acid
Approximate number of gel particles ml	100,000	100,000	200,000
Recommended indications	Folds, such as nasolabial folds, shaping facial contours, e.g. cheeks and chin lips, fullness and pouting	Wrinkles, such as glabellar, oral Lips: Fullness, pouting and vermilion border	Thin superficial lines, Such as worry lines, Periobital lines, Perioral lines,
Where to inject	Deep layer of dermis and/of surface layer of subcutis	Middle part of dermis	Upper part of dermis
Recommended degree of correction	100% No overcorrection	100% No overcorrection	100% No overcorrection
Volume of syringe	1ml 2ml	1ml	1ml
Needle size	27G	30G	30G

Products Compare

	Stabilized Hyaluronic Acid	Other Ingredients	Source	Syringe
Reyoungel Fine	24mg/ml		Non Animal	1.0ml
Reyoungel Derm	24mg/ml		Non Animal	1.0ml
Reyoungel DEEP	24mg/ml		Non Animal	1.0ml/2.0ml
Reyoungel Deep	24mg/ml			20ml/50ml/vial
Restylane	20mg/ml		Non Animal	1.0ml
Restylane Perlane	20mg/ml		Non Animal	1.0ml
Teosal	25mg/ml		Non Animal	1.0ml
Juvederm 24/30 HV	24mg/ml		Non Animal	0.8ml
Juvederm 24/30	24mg/ml		Non Animal	0.8ml
ReVanesse	24ml/ml		Non Animal	1.0ml
ReDexis	25mg/ml	25mg Sephadex		1.0ml
Hyaluderm	0		No Animal	1.0ml

Juvederm 24/30mg And Juvederm 24/30mg HV are registered trademark of Corneal Group.

Restylane, Restylane Touch, Perlane are registered trademark of HA AB INC.

Hyaluderm is registered trademark of LCA pharmaceuticals.

Teosal is registered trademark of Teoxane laboratories.

Revanesse is registered trademark of Prollenium Medical Inc.



Sources of Hyaluronic Acid

Superior Non - Animal Stabilized Hyaluronic Acid-gels Reyoungel Esthetics offers pure, Non-Animal Stabilized Hyaluronic Acid . This gel is the basis of all Reyoungel. Hyaluronic acid is a polysaccharide and is one of very few substances that it is identical in all species and tissue types. Hyaluronic acid also exists naturally in our bodies and its most important function is to carry and bind water . It plays an important role giving volume to the skin, shape to the eyes and elasticity to the joints . Nutrients , oxygen and hormones are able to pass freely between the gel particles and the tissue, leaving the skin natural and healthy looking. As the gel breaks down, water takes its place and when totally absorbed the gel disappears unnoticed from the body. It biodegrades naturally and safely.




Stabilized for long duration



Because without modification hyaluronic acid remains in tissue only a few days , Conversely, too much modification can negatively impact biocompatibility. Our HA gel is only stabilized to approximately 1% . This provides longer-lasting effects with minimal tissue disturbance. Nutrients, oxygen and hormones are able to pass freely between the gel particles, leaving the skin healthy during the entire residence period.

Before and After

Before

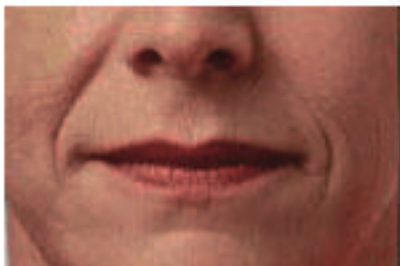

After

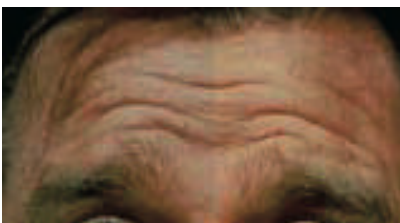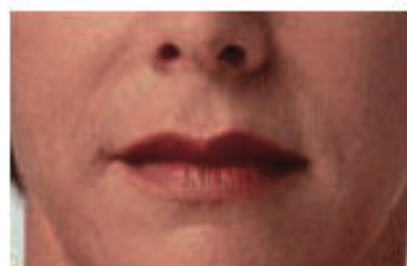



Reyoungel

Bioha laboratories

Address: 21 Floor Shengai Plaza ,No.88 Caoxi Rd., Shanghai City, China

Tel:+86 21 5169 8555

Fax:+86 21 5168 7555

Email: info@reyoungel.com

www.reyoungel.com

